

Assemble Your Shield4Dentists


1. Attach the Elastic Band
2. Peel the Protective Foil
3. Mount the Visor Onto the Frame
4. Adjust the Shield Angle





Tip: if you view these docs on a computer or a mobile device, you can double click the images to view them full-screen.

1. Attach the Elastic Band

Weave the band through the holes as you can see in the pictures below. This way, the band will remain adjustable, but will not slip. Adjust the length of the band to your liking.

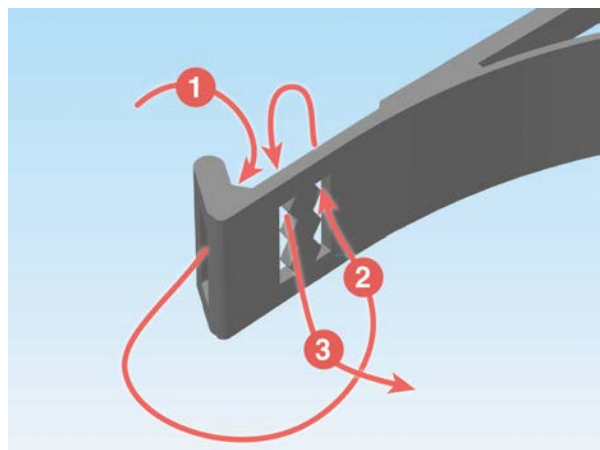

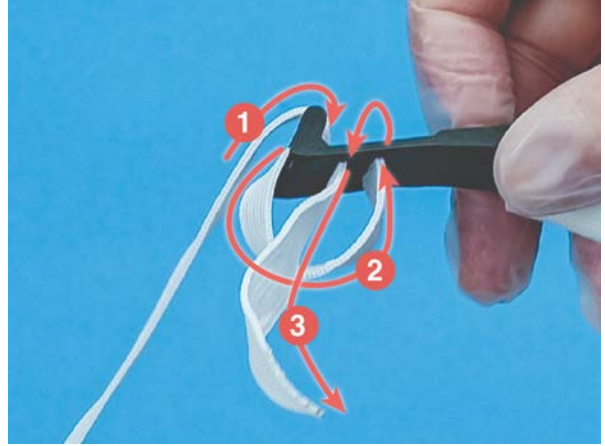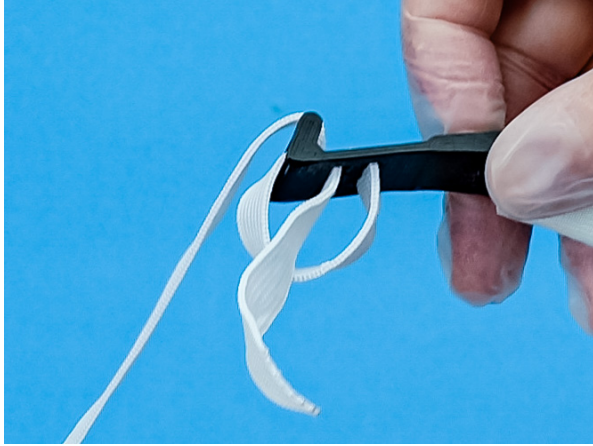

2. Peel the Protective Foil

The PETG visor comes with a protective foil on **both sides**.

If you don't remove it, you will not see very well.






Don't forget the 2nd side!



All clear!



3. Mount the Visor Onto the Frame

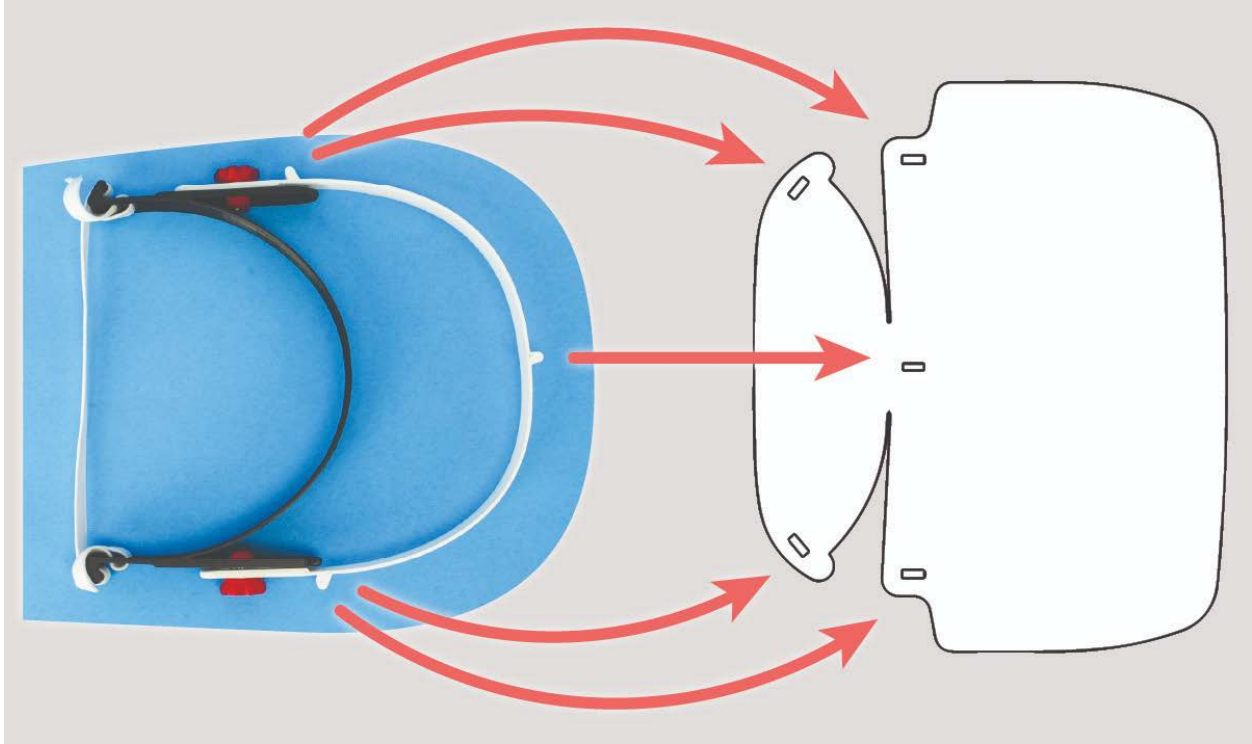



Start with the slot on one side and follow with the slot in the middle. These two should be easy.





Now, flex the second half of the frame inside, this making it possible to insert the frame into the 3rd slot in the visor.

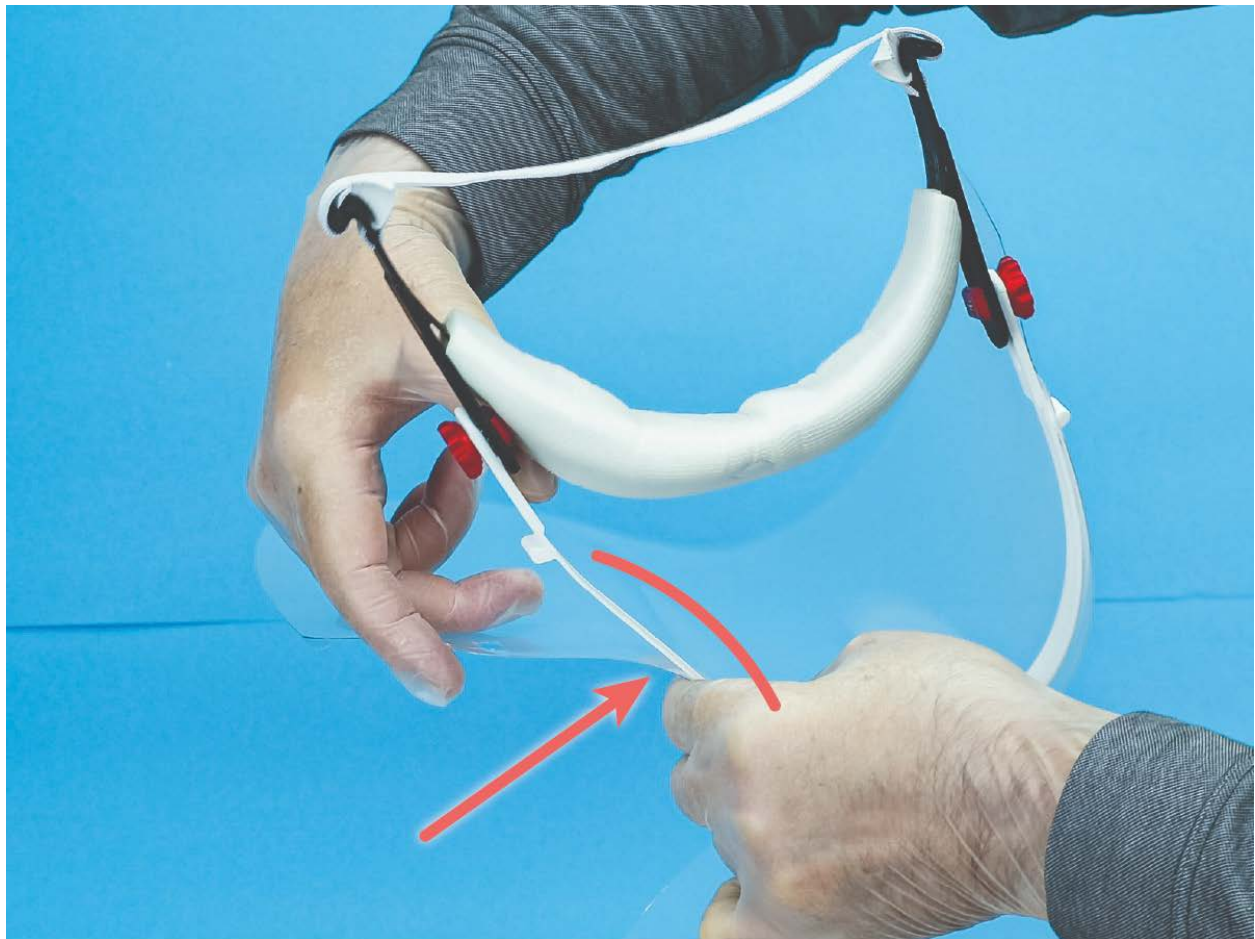


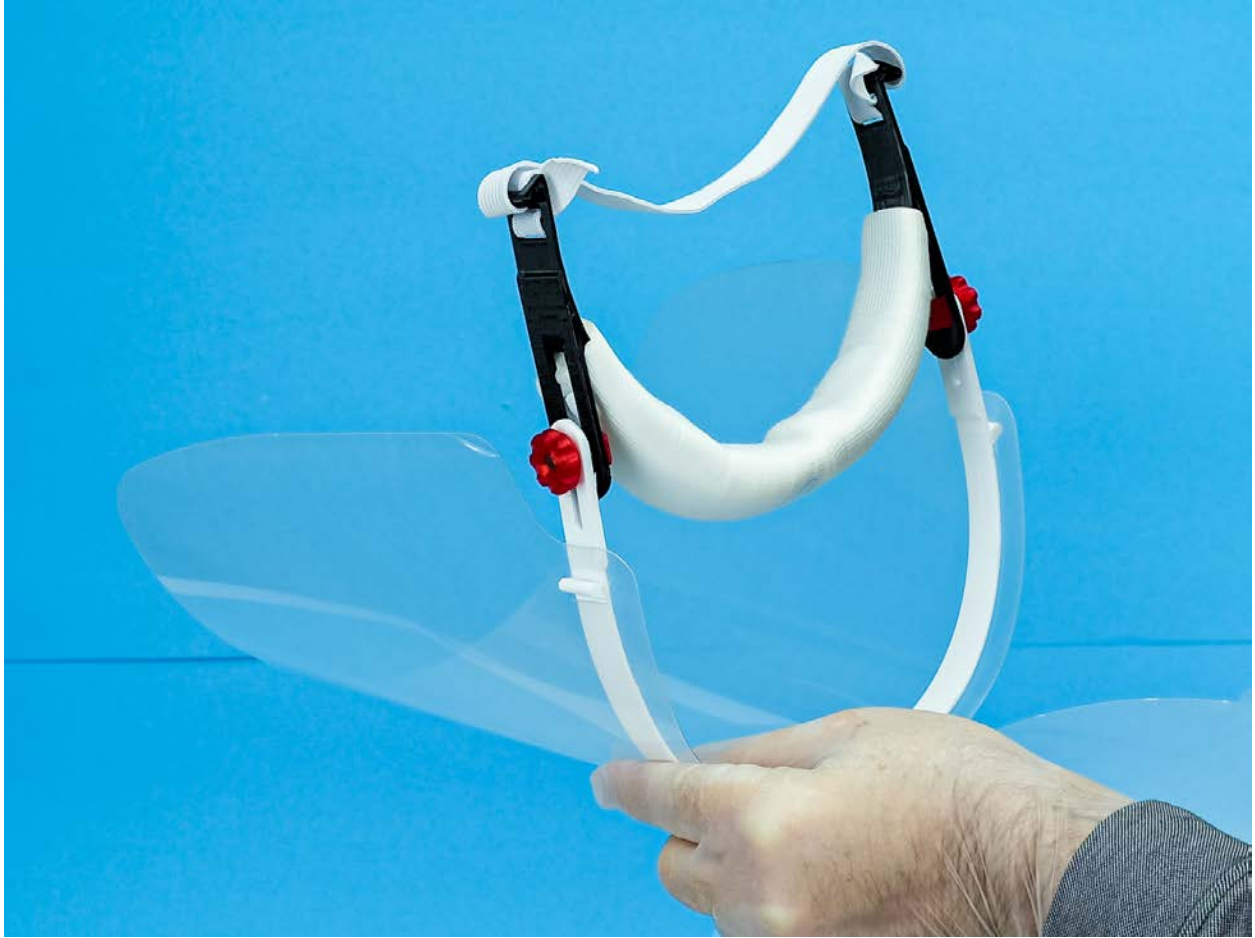

This is what the shield should look like at this moment.



Finally, latch the last two slots at the top of the visor to finalize the protective cover.

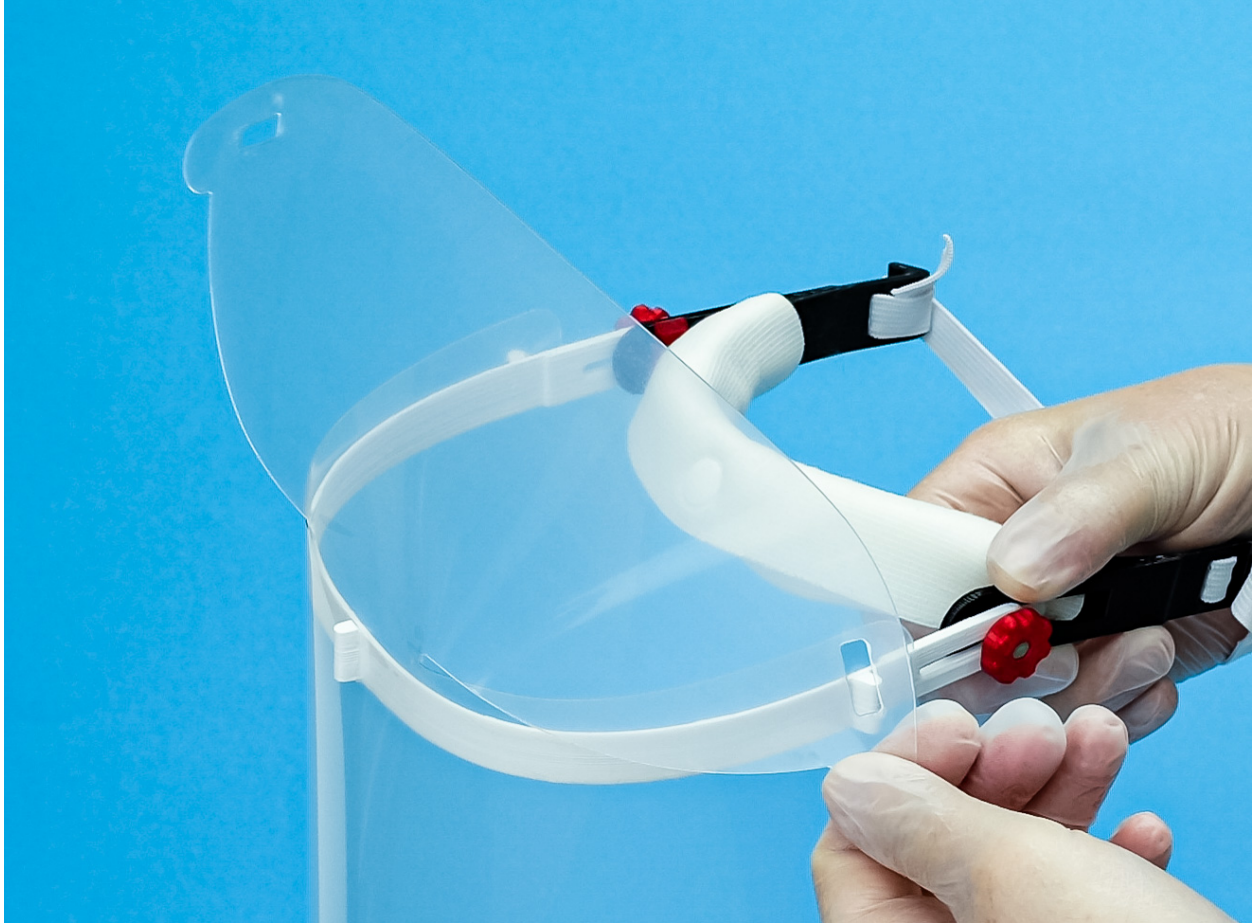


Done. Your shield is all assembled.



4. Adjust the Shield Angle

Loosen both of the adjustment knobs on the sides of the frame to adjust both the distance from your face and the angle.

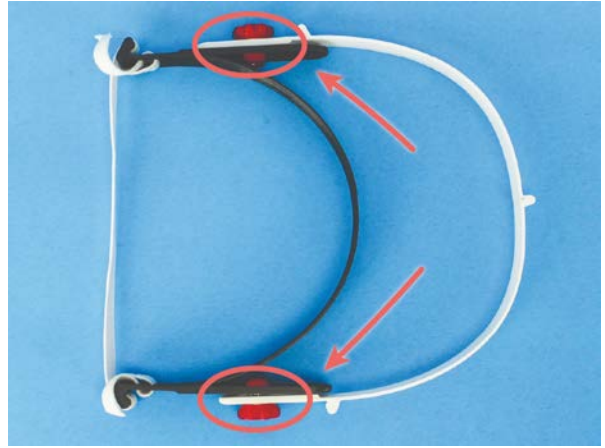

You may be interested in:

[← Go back to the Main Page](#)